

Fentanyl Test Kits

SW-0690





Read Before Use

As a precaution, gloves and eye protection should be worn when handling substances and using test units. Do not puncture any part of unit. Do not remove probe after breaking the ampule. In event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Remove t h e SWIPPERTM Cap from the test unit.



Ensure that the test tube remains upright in order to avoid the ampule falling out of the tube

Sample the suspected substance with SWIPPERTM

Sample multiple locations



Test Preperation

Return the SWIPPERTM Cap to the test tube. Press firmly.



S Do not continue unless

the SWIPPERTM Cap properly secured.

CAUTION

The kit contains an acid Make sure to follow all the safety procedures.

Identification

With thumb and forefinger. squeeze the test until the ampule breaks.



Gently shake test up and down to ensure that the liquid touches the Cap.



Results

The appearance of a red-purple (wine) color indicates the presence of Fentanyl.



The appearance of a reddish-orange color within one minute indicates the presence of Amphetamine or Methamphetamine.

This does not exclude the possibility that Fentanyl may be present.



The appearance of a purple color within one minute indicates the presence of an opiate such as Heroin, Morphine or Codeine.

* This does not exclude the possibility that Fentanyl may be present.



If none of the above colors develop, the above substances are not present in the tested sample.



Dilute the acid with some tap water before disposing the kit in the trash.





www.getswipper.com

Detection Solutions

Manufactured by IDenta Ltd Patented Technology • Made in Israel