

# SWIPPER™

DISCREET  
DRUG TEST

## Amphetamine/ Methamphetamine Test Kits

SW-5000




**Read  
Before  
Use**

As a precaution, gloves and eye protection should be worn when handling substances and using test units. Do not puncture any part of unit. Do not remove probe after breaking the ampule. In event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

**Test Preparation**

**1**

Remove the SWIPPER™ Cap from the test unit.

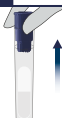

Ensure that the test tube remains upright in order to avoid the ampule falling out of the tube.

**2**

Sample the suspected substance or tablet with the SWIPPER™ Cap.

Touch or Press



**Test Preparation**

**3**

Return the SWIPPER™ Cap to the test tube. Press firmly.



**STOP**

**Do not continue unless the SWIPPER™ Cap is properly secured.**

**CAUTION**

The kit contains an acid. Make sure to follow all the safety procedures.

**Identification**


**4**

With **thumb and forefinger**, squeeze the test tube until the ampule breaks.



**5**

Gently shake test up and down to ensure that the liquid touches the Cap.

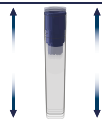

**Results**



The appearance of a reddish-orange color within one minute indicates the presence of **Amphetamine** or **Methamphetamine**.



The appearance of a purple color within one minute indicates the presence of an opiate such as **Heroin**, **Morphine** or **Codeine**.



The appearance of a dark purple to black color indicates the presence of **Ecstasy/MDMA**.



If none of the above colors develop, the above listed drugs are not present.



**Disposal**

Dilute the acid with some tap water before disposing the kit in the trash.



v2 01-22



[www.getswipper.com](http://www.getswipper.com)

Manufactured by IDenta Ltd  
Patented Technology • Made in Israel